


# Getting Started with the OneTouch Reveal® app




The OneTouch Reveal® app changes the way you see your blood sugar


- Sync data, seamlessly, from your **OneTouch Verio Flex®** meter directly to your smartphone
- Draws a timeline of important blood sugar events and activities
- **ColorSure™** technology transforms data into quick visual snapshots that connect your blood sugar with food, insulin and activity
- Helps you see and share progress with your family, friends, or your trusted care network

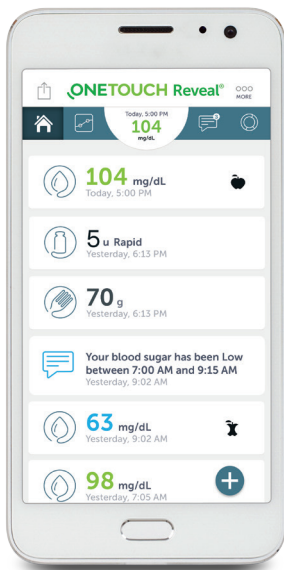

Timeline of important events

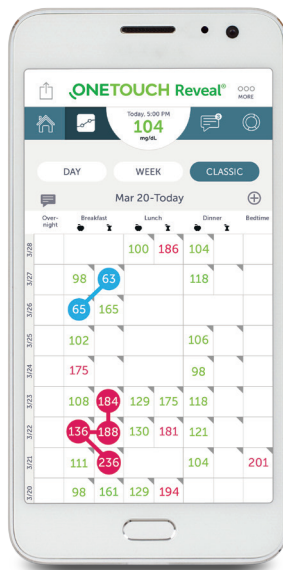

Visual logbook

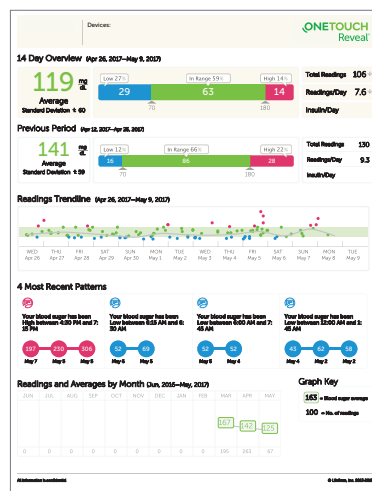

See and share Progress Reports




# Let's get started...


## 1. Download the FREE OneTouch Reveal® app

- Visit the app store on your mobile device, search "OneTouch Reveal", then install the app



## 2. Activate Bluetooth® on your OneTouch Verio Flex® meter

- Press the  button to turn your meter on
- Press the  and  buttons at the same time to activate Bluetooth®

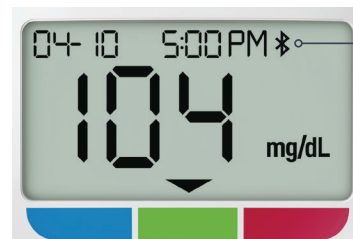


 **Bluetooth®**

Don't forget to also activate Bluetooth® on your mobile device.

## 3. Connect

- Open the OneTouch Reveal® app on your mobile device and follow the on-screen instructions
- When prompted, enter the PIN shown on your meter screen into your mobile device and tap **Pair**

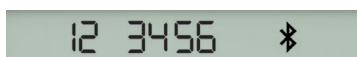

Example of a PIN number



You can also view your account on your computer at [OneTouchReveal.com](http://OneTouchReveal.com)

Now you're ready to test and sync!

Discover more at [OneTouch.com](http://OneTouch.com) or call 1 866 693-0599.